

Introducing the efficient way to collect blood samples

New!  
**TEGO**<sup>TM</sup>  
Card

## Collecting blood has never been this easy

### Simple and safe

- Easy to apply blood samples to the blood card
- Blood sample cards can be inserted into envelope immediately after blood collection and are ready to be sent for laboratory testing
- Makes sampling process quick and more efficient

### Compatible with existing lab processes

- TEGO<sup>TM</sup> is suitable for use in genetic/DNA testing such as genetic defects, trait selection, parentage and animal ID, as well as designated disease tests such as BVD and PRRS.
- DNA samples can be stored at room temperature for years without affecting sample integrity

### Cost-effective and convenient

- Allows for shipment of blood samples through regular postal mail—refrigeration is unnecessary
- Room temperature storage reduces costs of handling and storing blood samples



TEGO<sup>TM</sup> products



New!


# TEGO™ Card

Introducing the efficient way to collect blood samples


## Simplifies the process, reduces the risks

TEGO™ was developed in close consultation with U.S. laboratories, veterinarians and livestock producers worldwide.


### Quick and easy

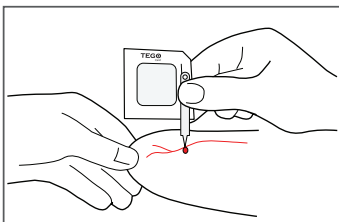


1. If blood is not readily available, use a new lancet to prick a vein on the animals ear or other appropriate location.

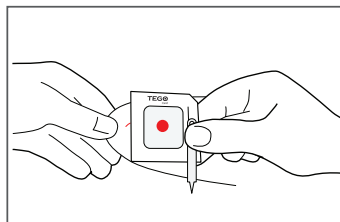

2. Holding the card by the frame, allow the blood to form a drop and blot onto the center area of the card.

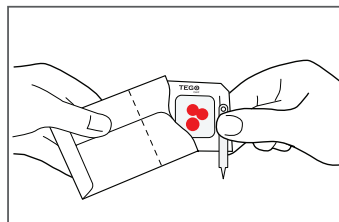

3. Place card into the envelope provided and label card and/or envelope as required.

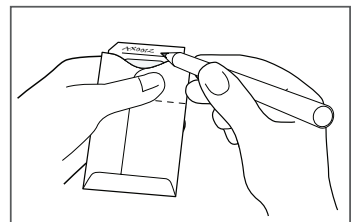

4. Fold over the open end of the sample envelope and place in a mailing envelope to send via postal mail. Safely dispose of the lancet or sharps device.

#### Available Blood Collection Kits

| Product Code | Description            | Qty/Pack | Qty/Carton   |
|--------------|------------------------|----------|--------------|
| A100915      | TEGO™ Card             | 1/pack   | 1,000/Carton |
| A100917      | TEGO™ Card with Lancet | 1/pack   | 1,000/Carton |
| A100919      | TEGO™ Card             | 10/pack  | 1,000/Carton |
| A100920      | TEGO™ Card with Lancet | 10/pack  | 1,000/Carton |

[www.itlanimalhealthcare.com](http://www.itlanimalhealthcare.com)

For sales enquiries, please contact:

#### ITL Animal Healthcare

(The Americas, Europe, Japan, Korea)  
Phone: 1-888-411-2851 (US Toll Free)  
Phone: +1 (703) 435-6700  
Fax: +1 (703) 435-6717

[sales@itlanimalhealthcare.com](mailto:sales@itlanimalhealthcare.com)

#### ITL Corporation

(Australia, NZ, Indonesia, Malaysia)  
Phone: +61 2 9209 4004  
Fax: +61 2 9319 3874  
[tego@itlcorporation.com](mailto:tego@itlcorporation.com)

TEGO is a trademark of ITL Corporation, Melbourne, Australia.  
©2011 ITL Corporation. All rights reserved. Patent pending.

There's security in knowing that you're providing the most advanced care for your animals

Single use only