
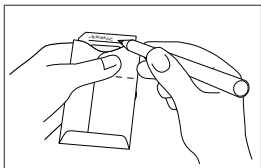
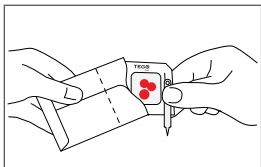
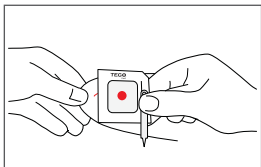


TEGO™ Card

Animal Blood Collection Kit

Instructions For Use

1. If blood is not readily available, use a new lancet to prick a vein on the animals ear.
2. Holding the card by the frame, allow the blood to form a drop and blot onto the center area of the card.
3. Place card into the envelope provided and label card and/or envelope as required.
4. Fold over the open end of the sample envelope and place in a mailing envelope to send via postal mail. Safely dispose of the lancet or sharps device.

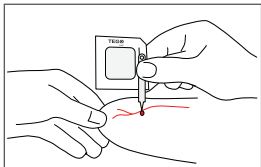

RM-03-04-2023 -REV-01

The TEGO™ Card Blood Collection Kit(s) facilitate collection, transport, processing and storage of animal blood samples for laboratory testing.

Precautions:

For Animal Use Only • Single Use Only • Keep Dry and Away from Sunlight • Do Not Use if Contaminated • Do Not Reuse Lancets - Dispose After Use • Consult Instructions For Use

Warnings:

- To allow cards to dry do not place blood samples in airtight containers or plastics bags.
- This product may contain sharps. Appropriate safety precautions should be taken during handling.
- Discard the kit according to appropriate procedure for disposal of biohazards waste.
- Blood collection from the ear in extremely cold conditions may be difficult due to reduced blood flow.
- Do not touch the interior sample area.

For enquiries please contact:

ITL Corporation

(Australia, NZ, Indonesia, Malaysia)

Phone: +61 2 9209 4004

Fax: +61 2 9319 3874

tego@itlcorporation.com

ITL Animal Healthcare

(The America's, Europe, Japan, Korea)

Phone: 1-888-411-2851 (US toll free)

Phone: +1 (703) 435-6700

Fax: +1 (703) 435-6717

itlanimalhealthcare@itlus.com

www.itlanimalhealthcare.com

TEGO™ is a trademark of ITL Corporation, Melbourne, Australia. Copyright© 2010 ITL Corporation All rights reserved